

Hip Hold 90 – Directions for Use

Mount the anterior support on the mounting stay, put the support cushion part against the stays and lock the connection. Lock it with the lower Bakelite knob by hand.

Then mount the anterior support on the operating table so that the lower support cushion can easily be moved towards the patient.

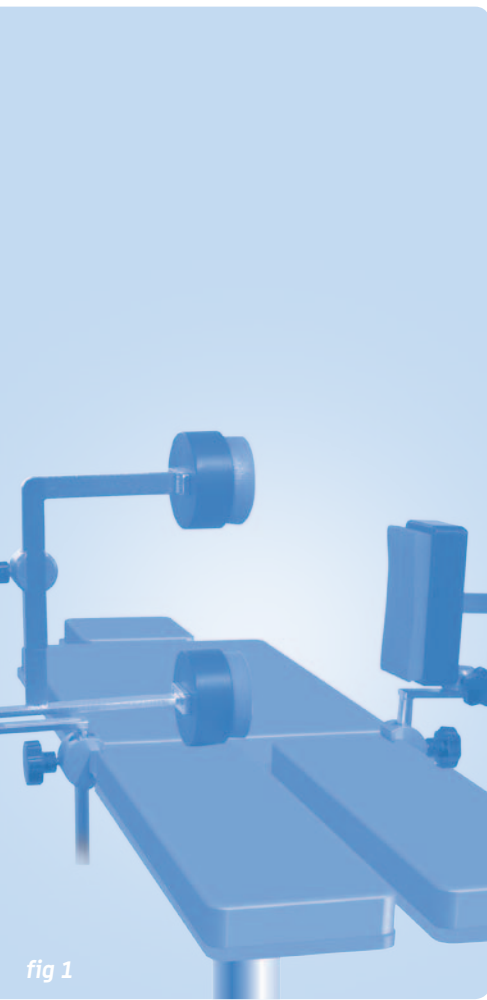


fig 1

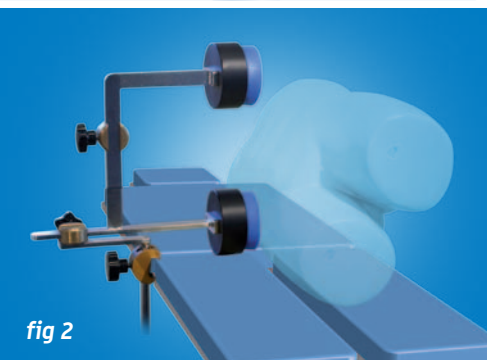


fig 2

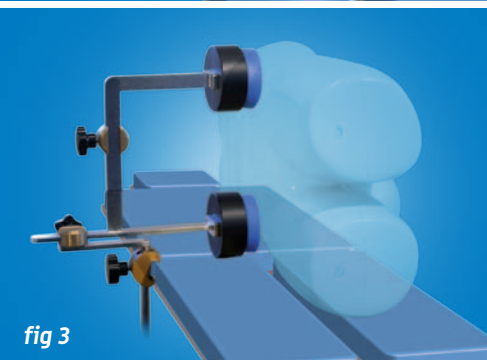


fig 3

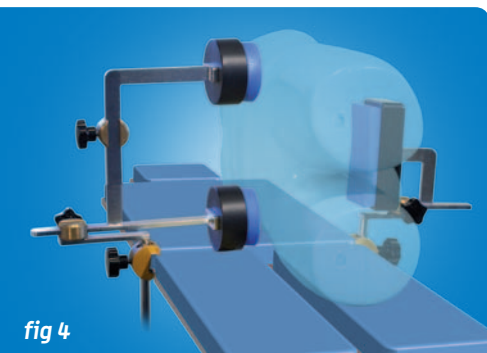


fig 4

1) Angle the patient slightly backwards and open the lower Bakelite knob so that the support cushion can be put against spina iliaca. Hand tighten the lower knob. Open the tablemount so that you can push the mount down to the operating mattress and push against the upper support cushion. **Lock the tablemount, open the lower Bakelite knob and push the lower support cushion carefully against the patient.** If you are using a soft type of operating mattress, with one hand push the mattress down under the support cushion to make it easier to put the cushion against the patient. Tighten the Bakelite knob.

2) Angle the patient against the upper support cushion, which will be placed against spina iliaca, and at the same time adjust the right height and tighten the Bakelite knob.

3) Keep the patient in position against the anterior support. Mount the superior support cushion which can be moved into the lock coupling from the outside. The superior mount should be placed against the sacrum – the height will be adjustable with the table coupling.

4) Verify that the patient lies stable fixed at 90 degrees against the table. If not the position has to be corrected by returning to step 1.

Notes

- If there is a risk of skin necrosis, additional cushioning should be applied to the support pad. The gel pad is ideal.
- Check that the support cushion and lock handle are fixed in the support arm.
- If any part of the Hip Fix shows signs of wear, it is essential that the part be replaced so that optimal results and absolute stability can be maintained.
- Cease use immediately if any parts appear faulty and contact the manufacturer.
- The Bakelite Knob should be tightened by HAND using moderate force.
- Check for any wear of the operating table attachments so that they do not negatively affect the use and mounting of the Hip Hold.
- Weight limit of patient 150kg.